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Hersteller

Usage instruction for impression.transfer, set.cap

vitaclinical is a business unit of VITA Zahnfabrik H. Rauter GmbH & Co. KG (referred to in the following as vitaclinical)
Manufacturer: VITA Zahnfabrik H. Rauter GmbH & Co. KG, Spitalgasse 3, 79713 Bad Säckingen, Germany

General

These instructions alone are not self-sufficient for the proper and safe use of the ceramic.implant system. The product is only to be used by dental surgeons with a detailed knowledge of dental implantology and experience in the use of implant systems. It is absolutely essential to follow the instruction manual (surgical.guide) and/or the instructions of an experienced operator. Training courses are offered by vitaclinical on demand.

Safety, liability and guarantee

The enclosed product is part of the vitaclinical implant system, and may only be used in conjunction with the corresponding original components and instruments as stipulated in the vitaclinical instructions and recommendations. The use of non-original, third-party components is detrimental to the performance of the implant, and leads to the loss of guarantee and replacement rights. Potential statutory liability claims for defects or product liability claims remain unaffected by this.

Instructions and advice regarding product application are transmitted in oral and written form, via electronic media or personal demonstration. This information, however, does not absolve the user from his or her personal responsibility to ensure that the product is suitable for its intended use, indications and procedures. (dieser Satz ist doppelt!)The use of the product shall take place outside of the sphere of influence of vitaclinical, and hence falls into the user's area of responsibility.

vitaclinical does not accept any liability for direct or indirect consequential damages resulting from incorrect usage of the product. The liability of vitaclinical for potential defects is subject to the terms and conditions of sale and delivery of vitaclinical.

Copyright and trademark

vitaclinical or any other products and logos of vitaclinical mentioned here are trademarks or registered trademarks of VITA Zahnfabrik H. Rauter GmbH & Co. KG.

Warnings and precautions

These instructions alone are not self-sufficient for the proper use of vitaclinical components. The product is only to be used by dental surgeons with a detailed knowledge of dental implantology and experience in the use of implant systems. Incorrect use can result in transfer errors which in turn require the prosthetic work to be repeated.

Product usage needs to be safeguarded

Product description

impression.transfer: accessory for transferring the implant position to the master model with the aid of an impression and a lab.replica.

set.cap: offers two other options for transferring the implant position to the master model.

1. With ceramic.implant CI, the shape of an individually milled abutment can be transferred. (Use of lab.shoulder rather than lab.replica in fabricating the model).
2. With ceramic.implant CII, the set.cap can be used to take an impression of an already cemented (and possibly corrected) ceramic abutment, in order to provide an accurate record of changes in the dimensions resulting from the cement gap.

Indications

The procedure using impression.transfer und lab.replica is suitable for single implants or bridge abutments that are sufficiently parallel and for abutments that have already been cemented in the case of ceramic.implant CII.

Use

To avoid transfer errors, ensure that both accessories (impression.transfer and set.cap) are correctly fitted on the ceramic.implant before taking an impression.

Packaging and sterility

impression.transfer and set.cap are supplied in sterile packaging and only intended for single use. Re-sterilisation is not possible.

Further information

Further information on using vitaclinical products is provided in the surgical.guide and the prosthetic.guide for ceramic.implant.

Explanation of symbols used

Order number

Batch designation

Sterile packaging, EO sterilised

Do not use in the case of damaged packaging

Not for re-use

Date of manufacture

Expiration date

Store away from sunlight

0124 The CE mark confirms that the requirements of the Council Directive 93/42 EEC concerning Medical Devices are fulfilled

Attention – please refer to the enclosed documents

Manufacturer